RIDDER | *Assistent*

CE

MONTAGEANLEITUNG
USER MANUAL

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Gebrauchsanleitung Rollator A130000

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
User manual for walker A130000



Dear customer,
thank you for choosing our product. Before use, please read the safety instructions carefully and keep the manual for further use. The manufacturer assumes no liability for incorrect assembly or improper use of the walker.

Delivery content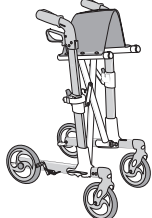

1x pre-mounted walker

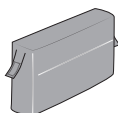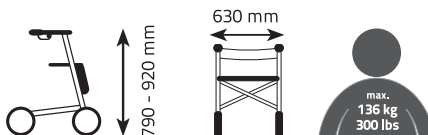1x fabric basket
(max. capacity: 9l, max. load limit: 5 kg / 11 lbs)

1x back belt

1x mounting instructions and user manual

Technical data:

Item no: A130000

Model: Walker IVO

Max. dimension of the mounted walker (w x h x d): approx. 630 x 920 x 770 mm

Smallest dimension of the folded walker (dismounted front wheels and accessories):
approx. 280 x 315 x 810 mm

Weight of the walker (without accessories): approx. 8 kg (17.6 lbs)

Max. user weight: 136 kg (300 lbs)

Max. load limit basket: 5 kg (11 lbs)

Seat height: approx. 570 mm

Seat dimensions: approx. 400 x 200 cm

Grip height: approx. 790 – 920 mm

i NOTE!

Please open the packaging carefully, so that the product does not get damaged. Check the full delivery content for completeness and intactness. If you find any defects, please contact the service address given.

1. Hand grip
2. Hand brake
3. Back belt
4. Frame (brake cables placed at the inner side)
5. Seat
6. Drawstring for folding
7. Fabric basket
8. Swivel wheel with reflectors
9. Locking lever

10. Walking stick holder
11. Screw for height adjustment
12. Snap-in aid for folding
13. Rear wheel with brake
14. Tipping aid

[i] NOTE!

Stability notes:

The walker supports an individual weight up to 136 kg (300lbs), INCLUDING the loading of the basket. The basket has a weight limit of 5 kg (11lbs).

 WARNING!	Warning, danger! Failure of respect the danger can result in serious injury or death.
 CAUTION!	Caution, danger! Failure of respect the danger can result in minor or moderate injury.
 NOTE!	Mind possible property damages

	Min. and max. height of the walker
	Max. width of the walker
	Max. user weight
	Can be used indoors and outdoors
	Follow the instructions!
	Contact details distributor
	Date of manufacture
	Declaration of conformity (CE conformity)
	Medical product
	Mind hand injuries
	Disposal of the packaging

Intended Use

This walker is used by people with limited mobility as a walking aid indoors and outdoors. It can also be used as a seat. The walker is only intended for private purposes and is not intended for use in the commercial sector. Use the product only as described in the instructions.

The recommended useful life of the product is a maximum of 5 years. This information assumes that the intended handling, care and safety instructions are observed.

Safety instructions

WARNING!

- The product is not designed for the use by persons with limited physical, sensory or mental abilities. It must not be used in case of a lack of knowledge or experience, unless instructions have taken place in which the risks were explained and the persons concerned are supervised by a caregiver.
- This walker is not a toy! Do not leave children use this product unattended.
- Do not use the walker under the influence of alcohol, medication or other drugs that affect your physical, mental and motor skills.
- All wheels must have constantly contact with the ground. This ensures the correct alignment of the walker. Therefore, do not use the walker on uneven grounds like sand or gravel. In particular, pay attention to uneven places such as gully tops. If possible, avoid driving over them as the wheels could get stuck.
- Do not use the walker on iced and snow covered or on very wet grounds. Slippery surfaces affect the slip resistance of the wheels in a negative way.
- Do not use the walker in bad visibility conditions (e.g. in fog or at night).
- If you use the walker outside, please use, if possible, the sidewalk. Wear striking and reflecting clothes.
- Do not use the walkers on stairs or escalators.
- Transport things solely in the basket provided.
Pay attention to the max. load weight and max. capacity.
- Do not use the walker as a vehicle while you are sitting on it. Before using the seat, the parking brake must be activated.
- When adjusting the height of the handles, make sure that you take up an upright and comfortable body posture.
In order to prevent bad posture, let yourself be instructed by a professional for medical and rehabilitation products.
- Make sure that the handles on both sides are set on the same height and that the adjustment screws are locked correctly.
- Before every use, check the function of the brakes and make sure that the side tubes and the seat are locked properly.
- Check regularly if the walker has damages or wear.
Have defective components replaced immediately and do not use the walker until it has been completely repaired. Do not make any changes to the product.

- Defective components must be replaced immediately and the walker must not be used until it has been completely repaired. Do not make any changes to the product.
- **Do not use the product if it is damaged!**
- Always respect the maximum load limits of the walker and its accessories.
Never exceed this limit, otherwise the function and stability of the product can not be guaranteed.
- Do not stand on the walker and do not use it as a stand-up or ascending aid.
- Since the walker is equipped with a folding function, there is always, and especially in the process of unfolding and folding, the risk of being bruised or trapped.
- Always take care to protect your fingers and your hands from being trapped.
- Risk of burns! Do not expose the walker to direct sunlight or other extreme heat sources as the material can heat up quickly.
- Keep packaging material away from children.

Commissioning



CAUTION!

Risk of injury from getting trapped during the entire assembly process!



The walker gets delivered in the folded state.

Unfold the walker: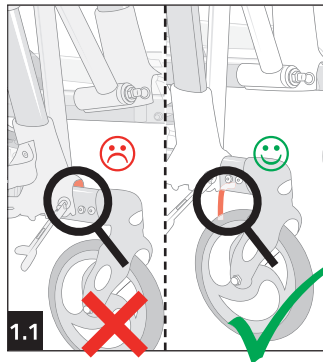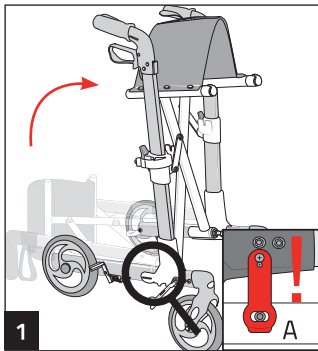



1 Put the two handle tubes in an upright position by loosening the snap-in aid (A) on the two side tubes.

1.1 Make sure that the position between the plastic hinge and the profile tube is correct.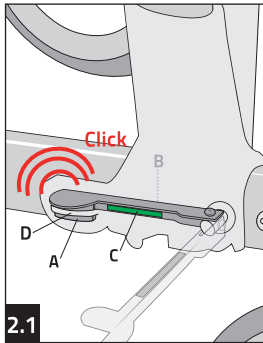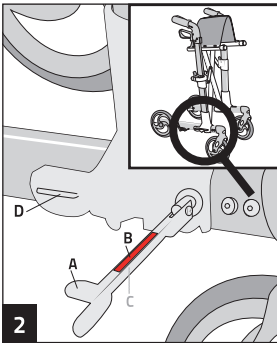

2 There is a locking lever with an insert flap (A) on the hinge. The lever has a red (B) mark on the outside and a green (C) mark on the inside. You can see the red one when the locking lever is open.


2.1 Move the locking lever. Turn it that way, that the inside (C) can be seen and the insert flap (A) can snap into the eyelet (D) provided on the hinge. Do this step on both handle hinges!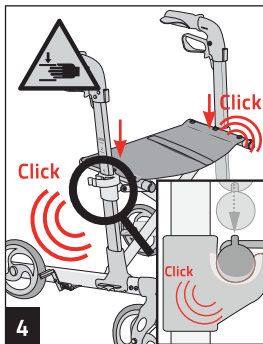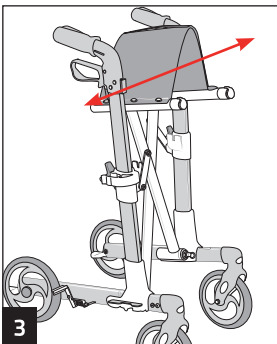


3 Take the handles of the walker and unfold them as far as you can.

4 The tubes of the seat must now be clamped into the brackets. Press the rods of the seat on the right and left down until they snap into the U-shaped brackets.


Fix the back belt: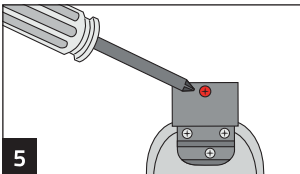


5 Loosen the two locking screws on the back belt.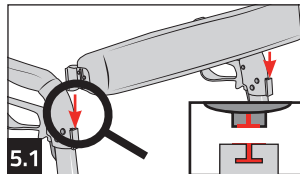

5.1 Now put the back belt on the device on the handles.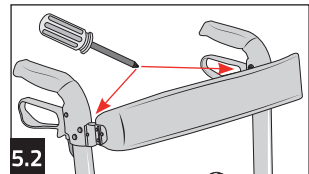

5.2 Now take the locking screws and screw them back into place.

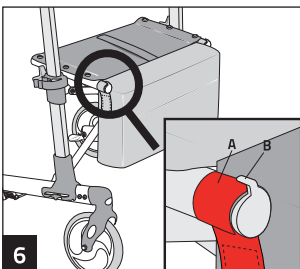Fix the fabric basket:

6 Velcro fasteners (A) are attached to the right and the left of the basket. Use the velcro fasteners (A) to attach the basket to the left and right tubes of the seat and then hook the resulting loop into the barb (B) attached to the tube to prevent the basket from slipping out.

Adjust the height of the handles:

i NOTE!

The height of the handles can be adjusted in 6 levels due to the screws. Assume an upright but comfortable body position with your arms hanging loosely. In this position, your hands should be able to grip the handles of the walker **1**.





! WARNING!

Incorrect adjustment of the handle height can lead to bad posture and other injuries **2** + **3**. Only use the walker with your correctly set handle height. Make sure that both handles are locked in at the same height.

For orientation, numbers are shown on the tube next to the height adjustment **4**. Turn the screw to loosen the handle **5** and snap it into place at a height that is comfortable for you. The handle must click into place at the same number on both sides **6**. Now the screw has to be tightened again **7**.

Please note that the handle must always be properly locked in one of the 6 heights provided for it before it is tightened.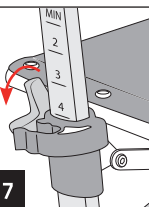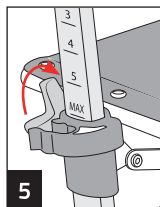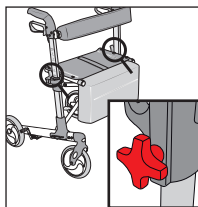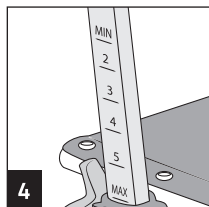

Correct use in the everyday life:

! WARNING!

To achieve the best possible support, hold the walker close to your body. Only then is a safe walk ensured. If you take the walker too far away from your body, it can roll away easily and you may not be supported adequately.

Use of the seat:

! CAUTION!

Only use the seat when the parking brake is activated. Do not being pushed while using the seat or push anyone else. There is a risk of falling! The seat must only be used by one person at a time. Do not use the walker if the ground is smooth or slippery.


Apply the brakes:

WARNING!

Always apply the right and left brakes at the same time so that they do not wear out unevenly. Check the brakes for correct setting and functionality before each use. The parking brake must always be activated before using the seat. Stop using the walker if the brakes can no longer be adjusted correctly.


The brakes must be readjusted from time to time due to wear.

If the braking performance decreases or is uneven, you have to re-tighten the brakes.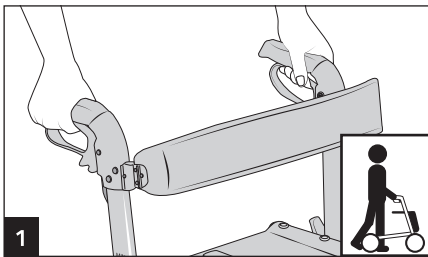

1 Hand-brake (while walking)



Pull both brake levers up towards the handbar at the same time and with the same intensity. To loosen it, just let it loose again. The harder you pull, the greater the braking power. If the braking power is too low despite firm pulling, the brake may have to be re-adjusted.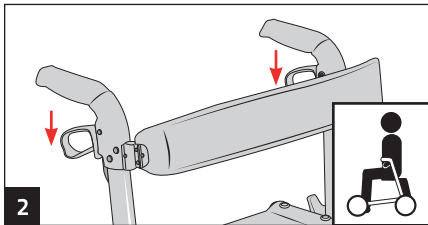

2 Parking-brake (while sitting)



Press both brake levers down until they lock and no longer snap back. Only then are the wheels fixed and the walker cannot roll away. To release, simply pull it back into the middle position with light pressure.

Folding: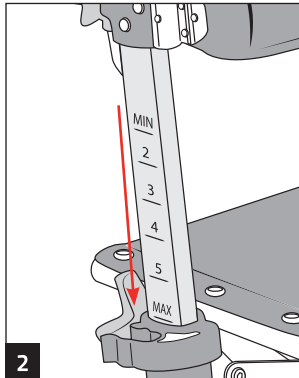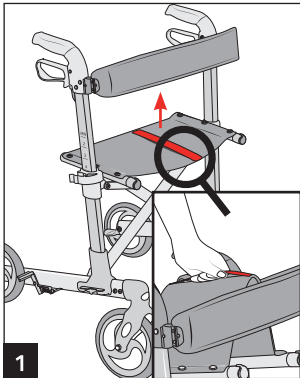

1 Grab the drawstring and pull the seat up, so that the tubes release from the holder and the walker begins to fold.

2 If necessary, set the handle height to the lowest level to save even more space.

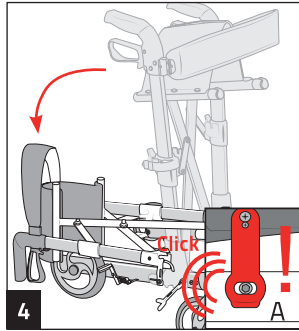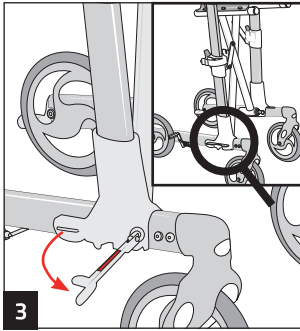3 Loosen the locking lever on the hinges of the two handle tubes so that the red mark can be seen.

4 Finally, fold the rollator from the upright to the lying position. Make sure that the frame engages in the snap-in aid (A). Then the rollator is secured and can be transported.

Storage:

For storage, the parking brake should always be activated.

Store the walker in a dry place by room temperature (approx. 20 ° C). Protect it from frost, direct sunlight or other weather influences. Keep the product out of the reach of children.

Cleaning and care:

Please note the cleaning and care instructions. Improper operation can damage the product.

Clean the walker gently with water and a standard commercial cleaner. Then dry the product with a cloth.

Do not use any aggressive cleaning agents and always make sure not to expose the walker to a hard water jet (e.g. from a high pressure cleaner), as this could damage small parts or the surface of the product.

Maintenance:

Before each use, check the functionality of the walker. The following parts require your special attention:

- Brakes
- Wheels
- Crew connections
- Overall appearance of the product
- Locks

Disposal of the walker:

Please dispose the product in accordance with the applicable regulations via an approved waste disposal company or via the municipal waste facility. If necessary, inform yourself about correct and environmentally friendly disposal.

Disposal of the packaging:

Please dispose the packaging environmentally friendly. Make sure that you separate the waste properly.

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RIDDER GmbH