RIDDER GmbH

Ridder Straße 1, 56379 Singhofen, Germany
Tel.: +49 (2604) 9541-0, Fax +49 (2604) 9541-50
Mail: info@ridder-online.de, www.ridder-online.de

Art.-Nr.: A1600007
2312021

1x


6x

6x



2x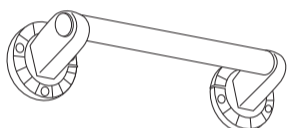

1x

A160300 A160401 A160601
A160300 A160400 A160600

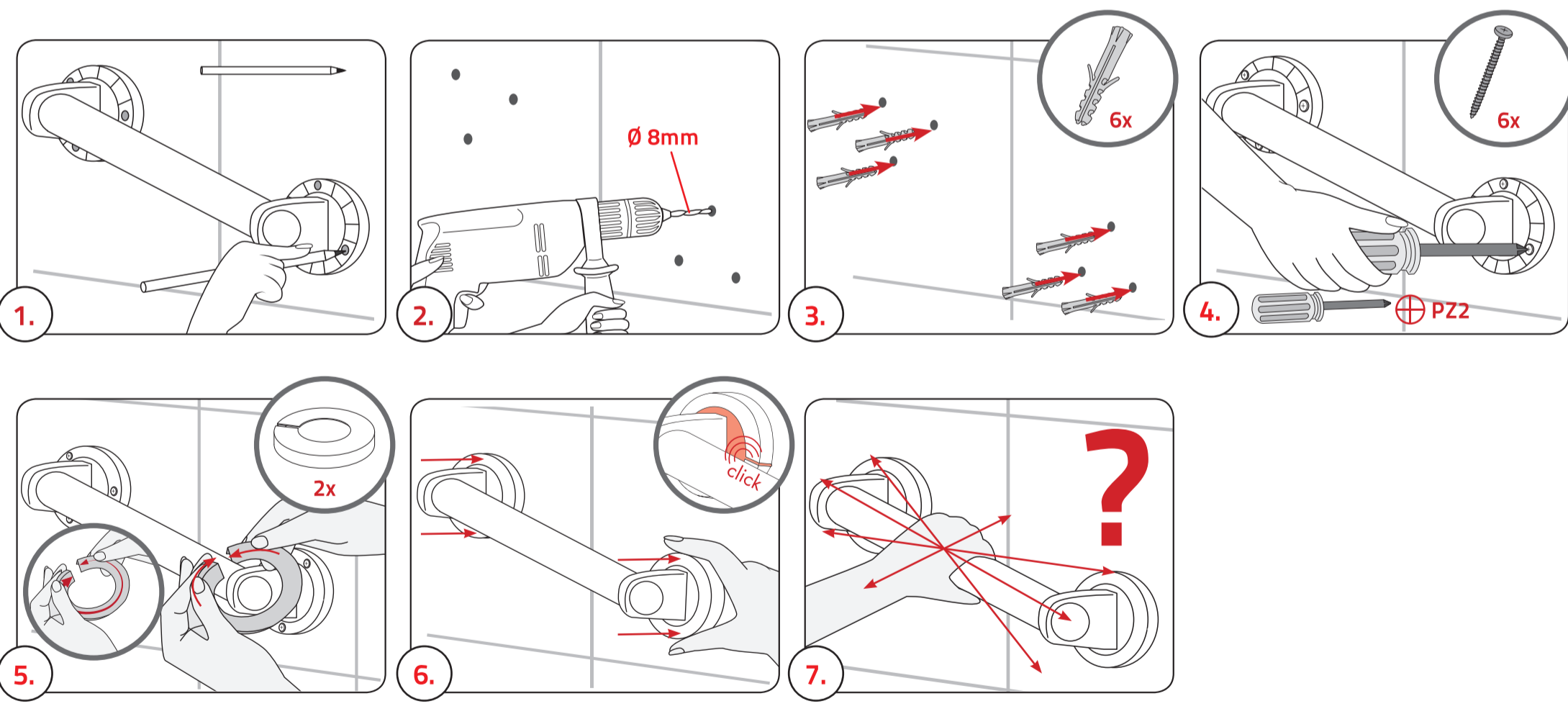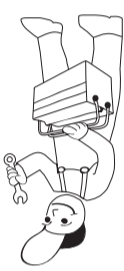1.

2.

3.

4.

5.

6.

7.

DE I HINWEIS!

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GB I NOTE!

Dear customer, please read the safety instructions carefully before use and keep the manual for further use. The manufacturer assumes no liability for faulty assembly or improper use of the product.

Safety instructions: This product is produced for applying in wet rooms and serves a stable body posture during a shower, a bath or by private using. Keep packaging materials away from children. Moist surfaces can impair the anti-slip function. The certificate does not contain tests for the use of people/patients who are in rehab or medical care sectors. The recommended maximum usage is 5 years. Do not use if the product is damaged.

Cleaning and care instructions: Please pay attention to the cleaning and care instructions. In the case of improper use the product can be damaged. Clean it carefully with water and standard commercial detergents. Subsequently, you can dry the product with a cloth.

▲ WARNING!

The installer must ensure that the construction of the wall guarantees a secure attachment of all wall mounting screws. The assembly must in any case be carried out on a stable, load-bearing wall with a smooth surface. Fasteners: dowels 8x40mm (0.31x1.57”), screws made of stainless steel 5x50mm (0.2x1.97”), suitable for reinforced concrete or solid brick M20. For other types of wall, a specialist firm of fastening technology has to be consulted.

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